

BLOCKING AND FINISHING

Eyezen™ Start design should be blocked and finished similar to a progressive lens:

- FRP (Fitting Cross 4mm above the plane of the micro circles) should be used for **blocking for finishing**.

Eyezen Start static cling verification decals (79900_PRO_ZEN) are available from Essilor.

VERIFICATION

Instructions to verify power in *Eyezen Start*

- 1 Verify the prescribed power at the distance verification circle
- 2 If prism was prescribed, then verify prism power at PRP
- 3 Confirm engraving to make sure *Eyezen Start* was provided



Transitions™


Crizal®

VARILUX®

Eyezen™

Xperio™

EYEZEN™ START CUTOUT CHART

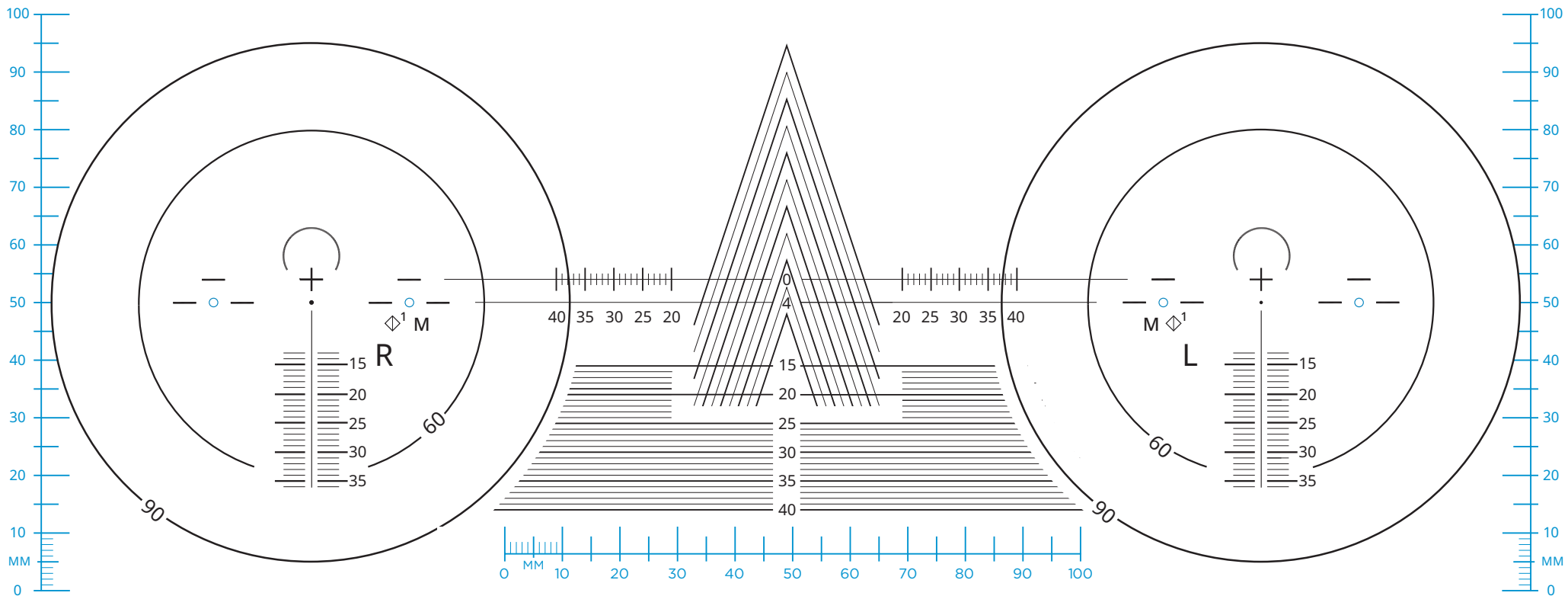

Please verify correct size with a ruler

LENS ENGRAVING SYMBOL LOCATION FOR MATERIAL	
MATERIAL	SYMBOL
Plastic 1.5	
TREXA® (Trivex®) 1.53	*
Airwear® Polycarbonate 1.59	P
Thin & Lite® 1.6	6
Thin & Lite® 1.67	π
Thin & Lite® 1.74	74

◇¹ Lens engraving symbol location for design

LENS ENGRAVING	
PRODUCT	SYMBOL
Eyezen Start	

PRINTING INSTRUCTIONS

To print this measuring chart at the correct size from Acrobat Reader:

1. Go to File, then select Print
2. Under Page Sizing & Handling options, Select Actual Size
3. Click on Print button to print.

Note: Chart measurements will be incorrect if "Fit" or other page-scaling options are selected. It is recommended to verify the printed chart is the correct size with a ruler.

Eyezen™
START